

Specimen Shipping Instructions

Bone marrow or whole blood

Sample Collection and Storage Requirements

To ensure optimal data analysis, all samples must be received the day following collection. Please follow these steps for proper sample handling:

1. Freeze a cold pack at least 12 hours prior to sample collection to maintain the necessary temperature during transport.
2. Sample Collection Requirements:
 - Bone Marrow: 1 x EDTA tube (purple top)
 - Blood: 3 x 10mL EDTA tubes (purple top)

Important: We are unable to accept shipments on Saturdays or Federal Holidays. Please schedule collection and shipments Monday through Thursday using FedEx First Overnight to ensure timely receipt of samples. Airbill affixed on Clinical Pak included.

Shipping Instructions


Please ensure all samples are properly labeled and stored according to these guidelines:

1. Parafilm the tube cap to prevent leakage.
2. Wrap each tube in absorbent materials (paper towels included).
3. Place the wrapped patient sample(s) in the bubble wrap envelope.
4. Place bubble wrap envelope and the completed **Test Requisition Form** in a biohazard bag.
5. Place biohazard bag and frozen cold pack in the insulated mailer bag. Zip seal, remove adhesive liner and seal the mailer bag. Place mailer bag in FedEx Clinical Pak (included).

The included airbill contains the relevant information for **FedEx First Overnight** shipping.

- **Shipping Address:** Predicta Biosciences
750 Main Street, Lab 413
Cambridge, MA 02139

6. Drop off at your nearest FedEx location.

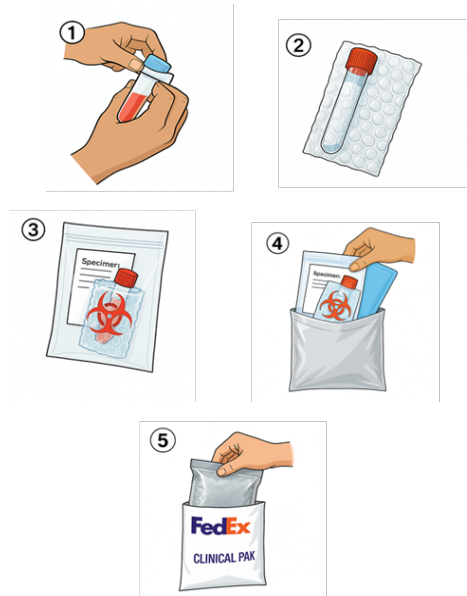

For Questions:

If you have any questions regarding the sample collection or shipping process, please contact us via:

Email: genopredicta@predictabiosciences.com

Phone: +1 (781) 691-3159